


Sol - M[®] Blunt Fill Needles

& Luer Lock Syringes with Blunt Fill Needles

- Blunt Fill Needles reduce the risk of needlestick injuries to healthcare workers when drawing up and preparing medication by removing the need for sharps.

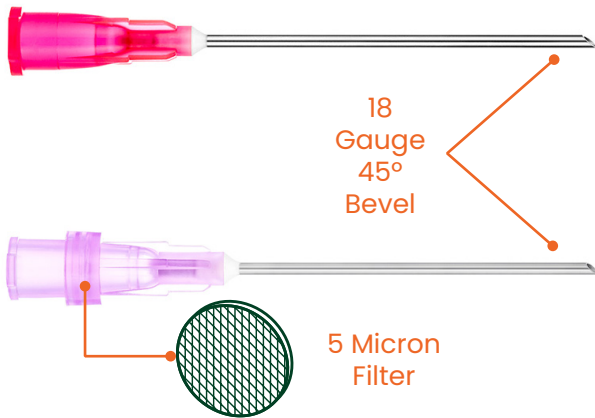

Features

- Eliminates the unnecessary use of sharps
- Color-coded hubs for easy identification
 - Blunt Fill Needle
 - Blunt Fill Needle w/Filter
- Needle hub is compatible with all Luer Lock and Luer Slip Tip syringes
- Available with 5-micron filter to prevent the drawing of glass particles when using ampoules (REF: 110022F)

SOL-M Blunt Fill Needles

Ref. Number	Product Description	Gauge	Needle Length	Box Qty	Case Qty
110021	SOL-M Blunt Fill Needle 18G x 1"	18 G	1"	100	1,000
110022	SOL-M Blunt Fill Needle 18G x 1 1/2"	18 G	1 1/2"	100	1,000
110022F	SOL-M Blunt Fill Needle w/Filter 18G x 1 1/2"	18 G	1 1/2"	100	1,000

SOL-M Luer Lock Syringe with Blunt Fill Needles

Ref. Number	Product Description	Volume	Gauge	Needle Length	Box Qty	Case Qty
1811815B	SOL-M 10mL Luer Lock Syringe w/ Blunt Fill Needle 18G x 1 1/2"	10 mL	18 G	1 1/2"	100	1,200
1831815B	SOL-M 3mL Luer Lock Syringe w/ Blunt Fill Needle 18G x 1 1/2"	3 mL	18 G	1 1/2"	100	800
1851815B	SOL-M 5mL Luer Lock Syringe w/ Blunt Fill Needle 18G x 1 1/2"	5 mL	18 G	1 1/2"	100	800



User Guidelines

● Blunt Fill Needle – for medication vials.

Insert the needle into the center of the vial stopper at 90° angle. This will help reduce the risk of rubber fragments from the side of the vial stopper contaminating the medication.



- The Blunt Fill Needle is for **single-use only**. Reuse will dull the needle, remove the silicone coating and increase the chance of coring the and rubber fragmentation.







● Blunt Fill Needle with 5 Micron Filter – for medication ampoules:





Draw up medication from glass ampoule. The 5 micron filter will aid preventing glass particles to be aspirated to the syringe.



● Blunt Fill Needle

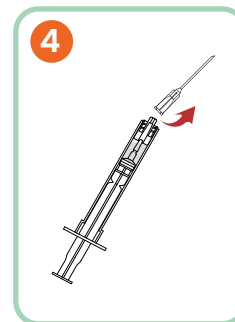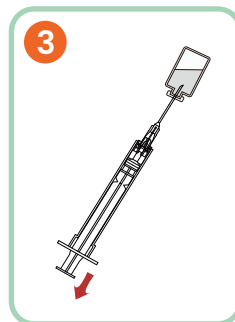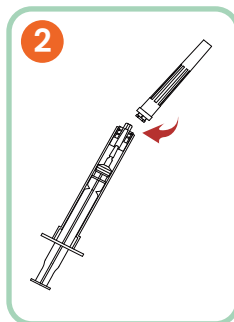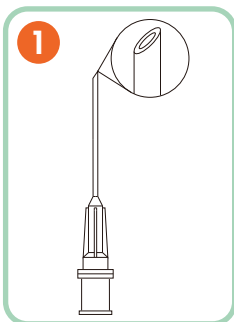

● Blunt Fill Needle with Filter

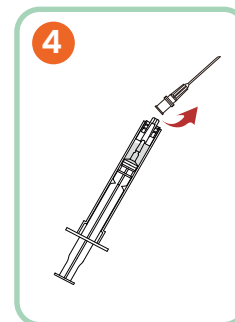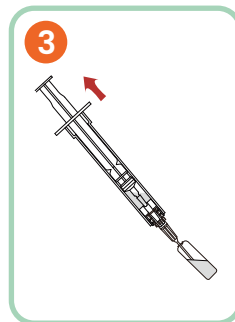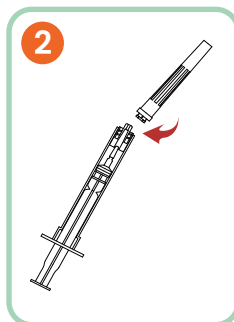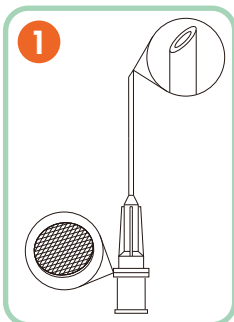

For more information, please contact:

📍 Sol-Millennium Medical Inc.
311 S Wacker Dr. Suite 4100
Chicago, IL. 60606

📞 833-637-0857
@ info@solm.com